



KONICA MINOLTA

June 11, 2020  
Healthcare Business Headquarters  
KONICA MINOLTA, Inc

## **COVID-19 Support from KONICA MINOLTA**

### Disinfecting for AeroDR NS (SKR4000)

#### **■ Product**

AeroDR NS (SKR4000)



#### **■ Available Disinfectants and Procedure**

##### **<Disinfection for AeroDR NS (SKR4000 Panel)>**

- Use ethanol or isopropanol for disinfection.
- Put the disinfectants on the soft and non-woven fabric, squeeze it well and wipe off the AeroDR NS (SKR4000) Panel surface. If you apply or spray the disinfectants on the AeroDR panel, the solution may enter into the equipment through exterior gaps and it may cause a failure.
- Be careful not to apply disinfectants onto the connectors and the LEDs.

#### **■ Caution**

- Turn off the power before performing disinfection.
- Using those disinfectants does not guarantee cleaning or disinfection efficacy and have not been tested by KONICA MINOLTA, Inc. for use with the COVID-19 virus. Please follow the policies and procedures of your institution for the cleaning and disinfection of equipment.
- To use those disinfectants, make sure to follow the instructions provided by each manufacturer.